


THUASNE®

en	Sport elastic wrist support	4
fr	Poignet élastique sport	5
de	Elastische Sport-Handgelenkbandage	6
nl	Elastische polsorthese sport	7
it	Polsiera elastica per uso sportivo	8
es	Muñequera elástica para uso deportivo	10
pt	Punho elástico para uso desportivo	11
da	Elastisk armbind sport	12
fi	Joustava rannetuki urheilukäyttöön	13
sv	Elastisk handledstöd sport	14
el	Αθλητικό ελαστικό περικάρπιο	15
cs	Elastická bandáž zápěstí sport	17
pl	Elastyczna orteza na nadgarstek sportowa	18
lv	Sporta elastīgā aproce	19
lt	Tamprus riešo įtvaras sportui	20
et	Spordi elastne randmetugi	21
sl	Elastičen zapestni trak šport	23
sk	Športový elastický náramok	24
hu	Rugalmas csuklórögzítő sportoláshoz	25
bg	Еластичен нажитник за спорт	26
ro	Orteză elastică pentru încheietura mâinii sport	27
hr	Elastični sport steznik za zglob	29

SPORT

Instructions for use - Instructions d'utilisation - Gebrauchsanweisung - Gebruiksaanwijzingen - Istruzioni per l'uso - Instrucciones de uso - Instruções de utilização - Anvisninger vedrørende brug - Käyttöohjeet - Råd vid användning - Οδηγίες χρήσης - Pokyny k používání - Szczegóły użytkowania - Lietošanas instrukcija - Naudojimo instrukcijos - Kasutusjuhised - Navodila za uporabo - Pokyny na používanie - Használati utasítás - Инструкции за употреба - Instrucțiuni de utilizare - Upute za uporabu

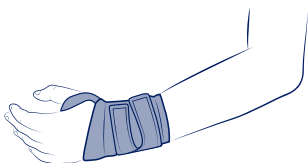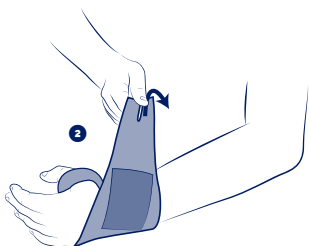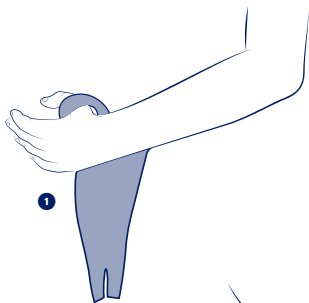en	Wrist circumference. Universal size
fr	Tour de poignet. Taille unique
de	Handgelenkumfang. Einheitsgröße
nl	Polsomtrek. Eén maat
it	Circonferenza del polso. Taglia unica
es	Contorno de la muñeca. Talla única
pt	Perímetro do punho. Tamanho único
da	Håndleddets omkreds. Kun én størrelse
fi	Ranteen ympärysmitta. Yksi koko
sv	Handledens omkrets. En storlek
el	Περίμετρος καρπού. Ένα μέγεθος
cs	Obvod zápěstí. Univerzální velikost
pl	Obwód nadgarstka. Rozmiar uniwersalny
lv	Plaukstu locītavas apkārtmērs. Universāls izmērs
lt	Riešo apimtis. Vienas dydis
et	Randme ümbermõõt. Üks suurus
sl	Obseg zapestja. Enotna velikost
sk	Obvod zápästia. Univerzálna veľkosť
hu	Csukló kerülete. Egy méret
bg	Обиколка на китката. Един размер
ro	Circumferința încheieturii. Mărimă universală
hr	Obujam ručnog zgloba. Univerzalna veličina

en

SPORT ELASTIC WRIST SUPPORT

Support level 1

Description/Intended use

This device is an elastic wrist orthosis providing joint support. The device is intended only for the treatment of the indications listed and for patients whose measurements correspond to the sizing table.

Composition

Polyamide - elastane - synthetic rubber (CR).

Properties/Mode of action

Adjustable fastening system.

Bilateral product.

Indications

Wrist sprain.

Wrist tendinopathies.

Contraindications

Do not apply the product in direct contact with broken skin.

Do not use in the event of known allergy to any of the components.

Do not use the product if the diagnosis has not been confirmed.

Precautions

Verify the product's integrity before every use.

Do not use the device if it is damaged.

In the event of discomfort, significant hindrance, pain, variation in limb volume, abnormal sensations or change in colour of the extremities, remove the device and consult a healthcare professional.

For hygiene and performance reasons, do not re-use the product for another patient.

Do not use the device in case of application of certain products on the skin (creams, ointments, oils, gels, patches...).

It is recommended to adequately tighten the device to achieve support without restricting blood circulation.

In the event of any modification in the product's performance, remove it and consult a healthcare professional.

In case of doubt before any sports activity, consult a healthcare professional.

Undesirable side-effect

This device can cause skin reactions (redness, itching, burns, blisters...) or wounds of various degrees of severity.

Any serious incidents occurring related to the device should be reported to the manufacturer and to the competent authority of the Member State in which the user and/or patient is resident.

Instructions for use/Application

Open the hook-and-loop fasteners.

Place the support on the back of the hand and insert the thumb. **1**

Tighten the support using the hook-and-loop fasteners. **2**

Garment care

Product can be washed in accordance with the conditions shown on this instruction leaflet and on the label. Hand wash. Close the self-fastening tabs before washing. Do not use detergents, fabric softeners or aggressive products (products containing chlorine, etc.). Do not dry clean. Do not tumble-dry. Do not iron. Squeeze out excess water. Dry flat. Dry away from any direct heat source (radiator, sun, etc.).

Storage

Store at room temperature, preferably in the original packaging.

Disposal

Dispose of in accordance with local regulations.

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©Thuasne - 2052901 (2024-03)
THUASNE UK Ltd
Thuasne SAS – SIREN/RCS Nanterre 542 091 186